READ CAREFULLY BEFORE USE!

EN	Instructions for Use
DE	Gebrauchsanweisung
FR	Mode d'emploi
ES	Instrucciones de uso
IT	Istruzioni per l'uso
SV	Bruksanvisning
NO	Bruksanvisning
DA	Brugsanvisning
FI	Käyttöohjeet
EL	Οδηγίες χρήσης
NL	Gebruiksaanwijzing
PL	Instrukcja użytkowania
PT	Instruções de Utilização
RU	Руководство по эксплуатации
ET	Kasutusjuhend
SL	Navodila za uporabo
CZ	Návod k použití
TR	Kullanma talimatları
HR	Upute za uporabu
HU	Használati útmutató
SR	Uputstvo za upotrebu
RO	Instrucțiuni de utilizare
AR	مادختس ال ا تاميل عت
KO	사용 설명서
JP	取扱説明書
ZH	使用说明
BG	Инструкции за употреба

# FlurAbsorb/FlurAbsorb-S

For latest Instructions for Use please access [www.sedanamedical.com](http://www.sedanamedical.com) or scan QR code

EN	Instructions for Use	3
DE	Gebrauchsanweisung	4
FR	Mode d'emploi	5
ES	Instrucciones de uso	6
IT	Istruzioni per l'uso	7
SV	Bruksanvisning	8
NO	Bruksanvisning	9
DA	Brugsanvisning	10
FI	Käyttöohjeet	11
EL	Οδηγίες χρήσης	12
NL	Gebruiksaanwijzing	13
PL	Instrukcja użytkowania	14
PT	Instruções de Utilização	15
RU	Руководство по эксплуатации	16
ET	Kasutusjuhend	17
SL	Navodila za uporabo	18
CS	Návod k použití	19
TR	Kullanma talimatları	20
HR	Upute za uporabu	21
HU	Használati útmutató	22
SR	Uputstvo za upotrebu	23
RO	Instrucțiuni de utilizare	24
AR	تعليمات الاستخدام	25
KO	사용 설명서	26
JP	取扱説明書	27
ZH	使用说明	28
BG	Инструкции за употреба	29

# FlurAbsorb/FlurAbsorb-S

## EN - INSTRUCTIONS FOR USE

### 1 Intended use

The FlurAbsorb gas filter is intended to remove volatile anaesthetic in operating rooms, intensive care units, including mobile and ambulatory treatment areas, independent of any kind of ventilator used. The FlurAbsorb should be used in combination with the standard holder or mount provided to enable proper operation of the device.

### 2 What is FlurAbsorb?

FlurAbsorb is a filter cartridge for removing halogenated hydrocarbons anaesthetic gases such as Isoflurane and Sevoflurane. The filter consists of a labelled container with various filter inserts, special activated carbon, a locked closure, vents to enable air flow and a connector which enables connection to the exhaust port of a ventilator.

### 2.1 Symbols

The following symbols are used in the Instruction for Use or on the product itself.

Symbol	Description
<b>NOTE!</b>	Indicates information important for optimal use of product
	Read the instructions for use carefully before use
	Do not use if damaged
<b>REF</b>	Catalogue Number
	This way up
<b>MD</b>	European Medical Device Regulation 2017/745

### 3. Transportation and Storage of the FlurAbsorb gas filter

Please follow the procedures below in order to maximise the performance of the FlurAbsorb Filter.

### 3.1 Transportation

- Ensure the dispatch carton/box is transported with the label on the top.
- The carton should not be put on its side or on its top.
- Open the carton from the top.
- Do not use the FlurAbsorb filter if the seal is damaged or broken.

**NOTE!** The arrows on carton/ box is indicating the correct position of the Carton /box.

### 3.2 Storage

- The dispatch carton/box should be stored with the label on the top.
- Store in a dry dust free environment.
- Do not store in the open air.
- Store between -5°C to +35°C.

### 4. Operation of the FlurAbsorb Gas Filter

Before using the filter, ensure that all components needed for use are available. These include:

- The FlurAbsorb Gas Filter.
- The standard mount.
- Accessories.

SEDANA MEDICAL offers an accessory kit with REF 26072 which includes:


- 22 mm flexible tube to connect the anaesthetic gas filter to the exhaust of the ventilator.
- 6mm O<sub>2</sub> connecting tube/silicone tube to connect a side-stream gas monitor with the anaesthetic gas filter.
- Adapter 22m/22f to be used for the 6mm O<sub>2</sub> tube for simultaneous waste of gas from the gas monitor and the ventilator.
- Adapter 32m/30f to fit the flexible tube on certain ventilator exhausts e.g. Hamilton Galileo, Maquet Servo-i, Siemens Servo.

### 4.1 Fitting the FlurAbsorb Gas Filter on a ventilator.

1. Attach the mount for the anaesthetic gas filter to a standard bar on the ventilator or close to the ventilator.

2. Place the anaesthetic gas filter in the mount and remove the protective seal cap.
3. Put the 22mm flexible tube on the exhaust of the ventilator. For some ventilators it may be necessary to use a 22m/30f adaptor.
4. Place the 22mm flexible tube on the anaesthetic gas filter. If a side stream gas monitor is being used, then use a 22m/22f adaptor with a secondary outlet to connect the 6 mm O<sub>2</sub> tube for the exhaust from the gas monitor.

### 4.2 Application with the AnaConDa / Sedaconda ACD.

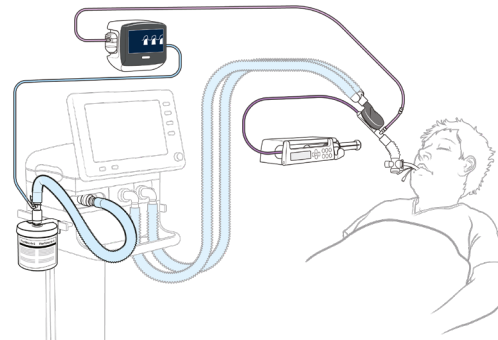### 5. Considerations during filter use

- FlurAbsorb (REF 26096) Filter should be changed when the storage capacity has reached 400g of Anaesthetic agent have been used.
- FlurAbsorb-S (REF 26094) Filter should be changed when the storage capacity has reached 125g of Anaesthetic agent have been used.

**NOTE!** A space is available on top of the FlurAbsorb (REF26096) lid to record both the start date of use and each syringe emptied.

### 6. Disposal of the FlurAbsorb Filter

- Put the protective seal cap back on the filter.
- Dispose of used filter according to hospital protocols.

## EN - INSTRUCTIONS FOR USE

### 7. Technical Data

	Standard (26096)	FlurAbsorb-S (26094)
Product Class	Class 1	Class 1
Temperature	Operation: +5 to +40°C Storage: -5 to 35°C	Operation: +5 to +40°C Storage: -5 to 35°C
Flow Resistance @10L/min	<10 Pa	<34 Pa
Height	23cm	13cm
Diameter	13cm	10.1cm
Weight	Approx. 1000g	Approx. 400g
Volume	2l	750ml
Storage Capacity	Approx. 400g	Approx. 125g

Product Description	REF Number
FlurAbsorb	REF 26096
FlurAbsorb-S	REF 26094
FlurAbsorb Accessory Kit	REF 26072
FlurAbsorb Mount	REF 26098
FlurAbsorb-S Mount	REF 26099

### 8. Warnings

- Caution when using active humidification with FlurAbsorb.
- Do not use FlurAbsorb-S with active humidification.
- Replace the filter immediately in case of occlusion
- In case of a serious incident please contact the competent authority and the legal manufacturer, Sedana Medical Ltd.

### Sedana Medical Ltd Address

Unit 2A, The Village Centre, Two Mile House, Naas, Co. Kildare, Ireland W91 PWH5

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